

BRUNO

Back-Saver™



Only
BRUNO
tests lift designs
to
three times
rated-weight
capacity!

Model
AWL-1600

- ◆ The Bruno Back-Saver Vehicle Lift is simple to operate and does not require any manual lifting. Just fold the chair up and roll it onto the platform. Turn the key to the ON position and depress the rocker switch to the UP position. Unloading the chair is just as simple and trouble-free.
- ◆ The Back-Saver is rated to handle most folding wheelchairs, with 22 in/56 cm or larger rear wheels, weighing 100 lb/45 kg or less.
- ◆ Bruno's Back-Saver will fit a standard Class I hitch tongue or a Class II/III if an adapter is used. It also features a strong, tubular frame for structural rigidity and long-term durability.
- ◆ The Back-Saver features a self-locking, Hold-Tite Arm that's designed to grip the chair firmly during transport, without marring its finish.
- ◆ In addition, the Back-Saver meets state and federal D.O.T. regulations for license plate illumination and visibility. It also includes an extra brake light for security and peace of mind.

OPTIONAL BRUNO POW'RPAC



- ◆ Easy to use!
- ◆ Convenient!
- ◆ Works in most minivans, SUVs, station wagons & sedans!

Pow'rPac easily installs in your vehicle – just plug into the vehicle's power outlet!

The complete, independent power supply for your automotive lift!

- ◆ Expert installation and complete, after-sale service is provided by Bruno's worldwide network of dealers.
- ◆ Only Bruno has over 20 different interior/exterior scooter, wheelchair and power chair lifts for nearly any sedan, minivan, sport utility vehicle (SUV), full-size van, pickup truck and crossover vehicles.
- ◆ All Bruno Lifts are eligible for reimbursement under the terms of Mobility Programs offered by GM, Honda, Hyundai, Saturn, Chrysler, Ford, Toyota, Lexus, Volvo and Volkswagen.

**BRUNO**
for your independence

Using the *Back-Saver*[™] is easy!

1



Fold the wheelchair and prepare to lower the platform.

2



Insert the key, turn to the ON position, and depress the rocker switch to the DOWN position.

3




Once the Back-Saver platform is lowered, simply roll the wheelchair into position.

4



With the wheelchair placed on the platform, depress the rocker switch to the UP position.

5

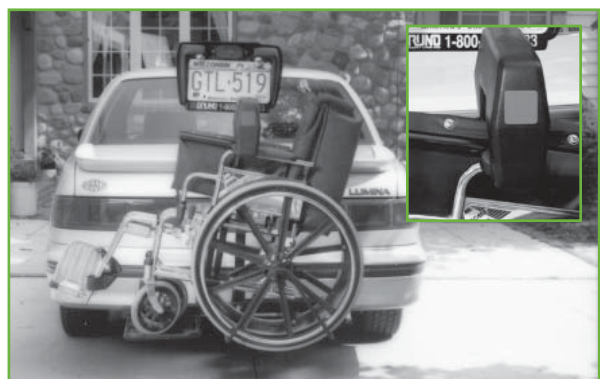

As the wheelchair is raised, it is gripped by a self-locking, Hold-Tite Arm.

6



The wheelchair is secured and ready for transport. Unloading is just as simple and trouble-free!



The Lift weight rating is the MAXIMUM load value. Factors inherent to specific applications including, but not limited to, vehicle structure and condition, operating environment, and customer ability may reduce the effective weight rating.

Represented By:



Bruno Independent Living Aids, Inc., 1780 Executive Drive,
P.O. Box 84, Oconomowoc, WI 53066

Toll Free 1-800-882-8183 ♦ 1-262-567-4990
FAX 1-262-953-5501 ♦ www.bruno.com